




activheal

WOUND CARE
BECAUSE YOU CARE

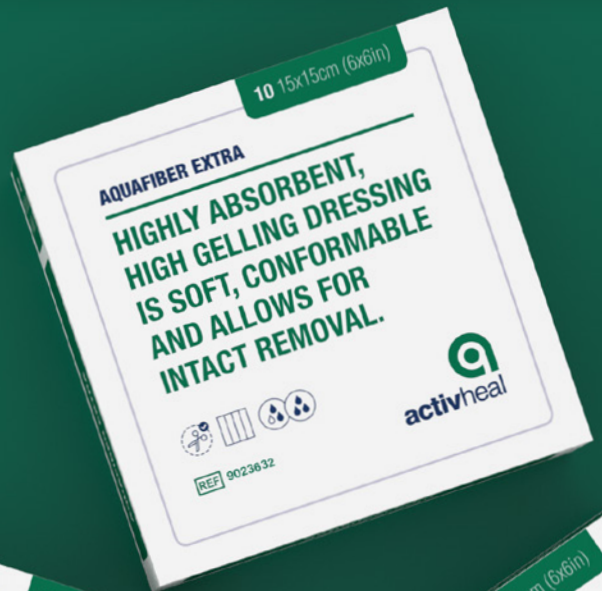

AQUAFIBER® EXTRA

WOUND CARE
BECAUSE YOU CARE

FEATURES

Highly absorbent // High gelling // The dressing can remain *in situ* for up to 7 days // Reduces risk of maceration // Maintains a moist wound healing environment // Aids autolytic debridement // High wet tensile strength //

ACTIVHEAL® AQUAFIBER™ EXTRA CREATES A MOIST WOUND HEALING ENVIRONMENT FOR MODERATE TO HEAVILY EXUDING WOUNDS.

ActivHeal® AquaFiber™ Extra is an innovative combination of two proven wound care technologies based on CMC and alginate fibres. A hidden integrity layer in the dressing supports the tensile performance.


ActivHeal® AquaFiber™ Extra is a soft, conformable, highly absorbent and non-woven dressing which turns into a soft, cohesive gel when in contact with wound exudate. Thus it helps with autolytic debridement and maintains a moist wound healing environment following the principles of advanced wound care. ActivHeal® AquaFiber™ Extra effectively addresses the key challenges of improved exudate management, gelling and tensile performance.

- ✓ HIGH ABSORBENCY¹
- ✓ HIGH WET TENSILE STRENGTH¹
- ✓ INCREASED VERTICAL WICKING¹

PERFORMANCE


High absorbency

The unique design and fibre engineering of ActivHeal® AquaFiber™ Extra allows the dressing to absorb exudate and transform into a cohesive gel.

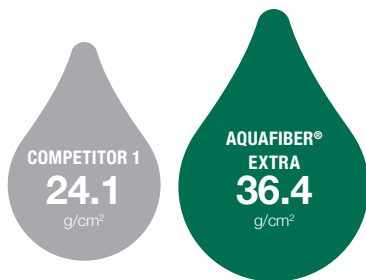


High wet tensile strength

The quilting pattern of ActivHeal® AquaFiber™ Extra is supported by a hidden integrity layer allows for intact removal of the dressing.

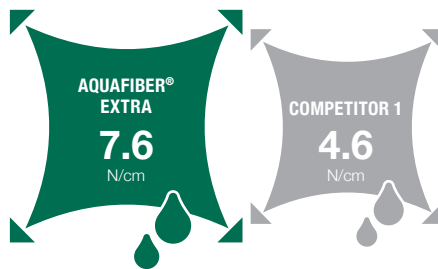

Increased vertical wicking

In-house analysis¹ has confirmed that the vertical wicking of the ActivHeal® AquaFiber™ Extra can be potentially helpful in the prevention of maceration within the peri-wound skin.

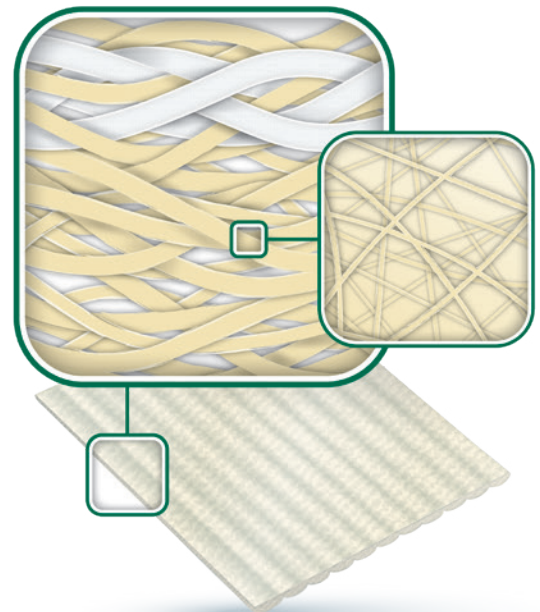

INDICATIONS

ActivHeal® AquaFiber™ Extra is indicated for the management of the following moderately to heavily exuding acute and chronic wounds:

- Pressure ulcers
- Venous leg ulcers
- Arterial leg ulcers
- Diabetic ulcers
- Graft and donor sites
- Post-operative surgical wounds
- Cavity wounds
- Superficial and partial thickness burns

Moderate > Heavy Exudate



Wound Types



SIZES AND CODES

ACTIVHEAL® AQUAFIBER™ EXTRA

SIZE (CM)	QTY PER CARTON	QTY PER CASE	PRODUCT CODE
5x5	10	10	9023625
10x10	10	10	9023632
15x15	5	10	9023649
20x30	5	10	9023656
2.5x30.5 Ribbon	5	10	9023663
2x46 Ribbon	5	10	9023670

Advanced Medical Solutions Ltd

Premier Park, 33 Road One, Winsford Industrial Estate, Winsford, Cheshire CW7 3RT
 T 08444 125 755 E activheal@admedsol.com W www.activheal.com



References 1. AMS Data on file LD024-16, LD037-16.

ActivHeal is a Registered Trademark of Advanced Medical Solutions.

© Advanced Medical Solutions 2021 // Non-branded competitor // AMS/AH-012-01